

Addresses

AUSTRALIA

Medartis Australia &
New Zealand Pty Ltd
Unit 6, 10 Hudson Road
PO Box 111
Albion Qld 4010, Australia
P 1300 858 853
P ^{int} +61 7 3326 8700
F 1300 854 665
F ^{int} +61 7 3862 2665

AUSTRIA

Medartis GmbH
Twin Tower
Wienerbergstrasse 11/12a
A-1100 Wien
P +43 1 99460 6499
F +43 1 99460 6498

FRANCE

Medartis S.A.R.L.
Domaine d'Entreprises
29, rue Condorcet
F-38090 Vaulx Milieu
P +33 474 99 94 14
F +33 474 99 00 19

GERMANY

Medartis GmbH
Am Gansacker 10
D-79224 Umkirch
P +49 7665 98 24 0
F +49 7665 98 24 10

MEXICO

Medartis S.A. de C.V.
Av. Presidente Masaryk 111 Piso 1
Col. Chapultepec Morales
Del. Miguel Hidalgo
MEX-11560, México, D.F.
P (+52 55) 3300 6054
F (+52 55) 3300 6006

NEW ZEALAND

Medartis New Zealand Ltd
PO Box 28461
Remuera
Auckland, 1541
New Zealand
P 0800 548 001
P ^{int} +64 9 909 0416
F 0800 548 002
F ^{int} +64 9 909 0419

POLAND

Medartis Sp. z o.o
ul. Sokolnicza 5/29
PL-53676 Wrocław
P +48 71 359 56 18
F +48 71 359 56 15

SPAIN

Medartis SL
Avda. de la Industria, 4
Edificio O, Local C
E-28108 Alcobendas, Madrid
P +34 91 661 33 15
F +34 91 661 33 90

UNITED KINGDOM

Medartis Ltd.
Batley Business Park
Suite 63, Annexe 4,
Technology Drive, Batley
West Yorkshire, WF17 6ER
United Kingdom
P +44 (0)1924 476699
F +44 (0)1924 472000

USA

Medartis Inc.
127 W. Street Rd, Suite 203
Kennett Square, PA 19348
USA
P +1 610 961 6101
Toll free 877 406 BONE (2663)
F +1 610 961 6108

Medartis AG
Hochbergerstrasse 60E
4057 Basel / Switzerland
P +41 61 633 34 34
F +41 61 633 34 00

www.medartis.com

WRIST-0500001_v0 / © 02.2010, Medartis AG, Switzerland. All technical data subject to alteration.

medartis®

PRECISION IN FIXATION

PRODUCT INFORMATION

Traction System

APTUS®
Wrist

The Traction System

A helping hand during surgery

The Traction System is used for wrist arthroscopy, surgical interventions of the wrist, carpals and metacarpals as well as for reduction of fractures. Depending on the indication, it allows for procedures with or without traction.

The Traction System securely holds the hand in the required position and traction, offering valuable support during surgery.

→ See Step-by-Step Instructions at www.medartis.com/meta/downloads

Characteristics

- Horizontal or vertical application possible
- Eccentric positioning of the hand for interference-free intraoperative radiographic monitoring
- Radiolucent arm support
- Easy assembly and disassembly
- Traction from 0 to 10 kg



Art. No. A-9110

Ordering Information

Wrist Traction System

(without Finger Traps)

Art. No.	Pieces per Pack
A-9110	1



Finger Traps, Steel



A-9112.01



A-9112.02



A-9112.03

Art. No.	Description	Pieces per Pack
A-9112.01	S, small	1
A-9112.02	M, medium	1
A-9112.03	L, large	1

Finger Traps, Nylon



A-9113.01



A-9113.02



A-9113.03

Art. No.	Description	Pieces per Pack
A-9113.01	green, small	1
A-9113.02	red, medium	1
A-9113.03	white, large	1

Velcro Strap




Art. No.	Length	Pieces per Pack
A-9114.01	450 mm	1
A-9114.02	550 mm	1

Clinical Cases

Case 1 – Vertical Application: Distal Radius Fracture

Dr. P. P. Borelli, Brescia (Italy)

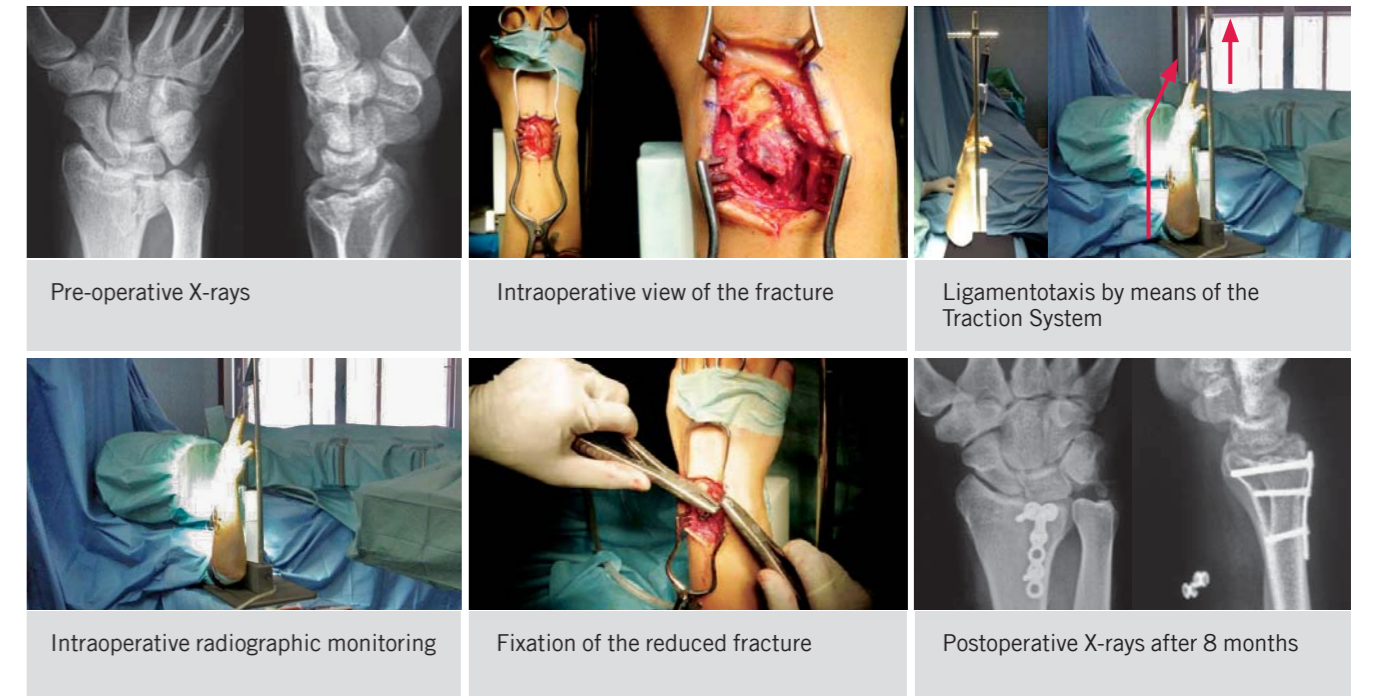

Case 2 – Horizontal Application: Distal Radius and Scaphoid Fracture

